Rollator JAGUAR

Art. 312410, 312411, 312412, 312413 & 312415

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione
Installations- och bruksanvisning

Indoor rollator
Champagne
7" black wheels

Jaguar standard
Blue, orange & purple

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

This Mobilex products is a medical device which is intended to be used only as a walking assistant by patients that because of invalidity are unable to stand, walk independently and who require additional stable support during walking. It gives for such persons the possibility to fulfil most of their regular, daily activities without the assistance of a helper. Any other use is prohibited.

Indications: The device is indicated for use as walking and standing support by persons who suffer from different diseases in the human motion system. The variety of such diseases is very wide but the symptoms are similar: people who because of their health status are unable to walk or stand are indicated to use rollators. The final decision to choose the correct type of device depends on the condition of the individual patient and should always be taken by a professional physician or physical therapist

If you have any questions, please contact your dealer or Mobilex A/S.

| | |
|--|--|
| Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark | Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk |
|--|--|

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used indoors on a flat and stable and surface.
- All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure that the rollator is properly balanced and should avoid accidents.
- When using the rollator in a stationary position, the hand brakes **MUST** be locked.
- The device must not be loaded with more than the max load value described in Technical Data and on the product's label.
- The basket has a load limitation of 5 kg

3. Description

Product description

The Mobilex rollator "Jaguar" has a powder coated aluminium frame It offers great manoeuvrability, stability and it has height adjustable push handles to fit the user. The basket eliminates the need to hand carry items.

The rollator is supplied in a carton with the following content:

- ✓ 1 Main frame with wheels, a basket with a front pocket with zipper and a tray
- ✓ 1 User manual

Assembly description

- Take the rollator out of the carton and place it upright on the floor.
- Compare with the list above if anything is missing. If you'd discover at that stage that some part is missing or damaged, do not continue the assembly process. Contact your supplier of the product right away.
- Support the handle with one hand and with the other hand grab one of the rear frame legs and gently pull it in backwards.
- Gently press the handle down until the frame is fully opened.
- The rollator is completely open when you hear a clear "click".
- Put the main frame with wheels on the flat surface. All wheels should touch the floor.
- Always ensure that the frame is stable and fixed when the rollator is opened.
- Adjust both push handles to the same height (see also function description).
- Ensure that the red snap buttons are securely locked so that the rollator is stable and parts can not move when forced.
- Always test to see that the rollator and attachments are properly and safely locked in place BEFORE use
- Failure to follow these warnings can result in personal injury.

Function description

- Adjust the height of the push handles so that when the user's arm is down along the body, the handle is at wrist height. Adjust the height by releasing the snap buttons located on upper frame tubes. Then adjust the handle to your desired height, close the snap button again securely. Repeat the steps for the other handle, so that both are at the same height.
- Care should be taken to ensure that ALL adjustments are secured and that castors and moving parts are in good working order before use.

Braking

- To stop the rollator simply pull the brake handle up and hold brake handle in the "UP" position to remain stationary. TO MOVE simply release the brake handle.
- To lock the brakes in "Engaged" position for longer period of "parking" push the brake handle up and fix it with the red C-cap To unlock the brakes pull the brake handle UP again and release the red C-clip by pulling it gently to the front
- While using the rollator in a stationary position, the hand brakes MUST always be locked. You should always pull the brakes when you don't plan to move the device. Never leave device on slopes without brakes pulled!

Folding

- Take away the tray from the rollator
- Press the folding button on the frame folding bar and push the rear frame leg gently toward the front leg to fold the rollator

Transport recommendations

The rollator is an easily foldable product convenient for transportation. If it is necessary to transport it, we recommend to fold it as it is written in the description above. The rollator should be transported in a way that is stable and with no risk of sudden and dangerous movements of the transported device.

Accessories

There are no accessories provided for this device

4. Technical data

| Rollator Jaguar | Orange Item nr. 312410 | Light blue Item nr. 312411 | Champagne Item nr. 312412 | Lilla Item nr. 312413 | SUPREME Item nr. 312415 |
|---|----------------------------------|-------------------------------|------------------------------|--------------------------|----------------------------|
| Height of the push handles (adjustable) | 83,5 - 94,5 cm | | | | 83,5 - 94,5 cm |
| Size of the folded rollator (H x W x D) | 83,5 x 54,5 x 27 cm | | | | 83,5 x 54,5 x 27 cm |
| Width | 54,5 cm | | | | 56,5 cm |
| Length (depth) opened / folded | 66 cm / 27 cm | | | | 66 cm / 27 cm |
| Weight | 6,3 kg | | | | 6,2 kg |
| Max. load (rollator / bag) | 100 kg / 5 kg | | | | 100 kg / 5 kg |
| Brakes | 2 lockable with parking function | | | | |
| Wheel sizes | Front and back: 7" (175 x 27 mm) | | | | |

Product label

The product label is located on the lower frame tube.

Sample of the product label for Jaguar 312410

| | | | |
|--|-------------------------------|--|--------------------|
| | QR-code link to homepage | | Manufacturing date |
| | Manufacturer | | Max load |
| | Observe the user manual | | Max width |
| | To be used indoor and outdoor | | serial no |
| | CE marked product | | Item Number |
| | Max load basket | | Medical Device |
| | Only to be used indoor | | |

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

| <i>Problems</i> | <i>Possible Causes</i> | <i>Solutions</i> |
|---|--|---|
| <i>The rollator is difficult to guide or push</i> | Dirt between wheels and forks | Remove dirt or hair between wheels and forks |
| <i>The brakes do not work or work only in one side</i> | The brake shoes are worn, or the brakes are not adjusted correctly | Adjust the brakes. Replace the brake shoes if needed (→ service centre) |
| <i>The seat does not fit in when the rollator is unfolded</i> | The locking device is in a wrong position or defective | Check that the locking device is in a correct position. Replace it if needed (→ service centre) |

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformit tserkl ring / Konformit tserkl ring / Dichiarazione di conformit )

Mobilex A/S, Gr nlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Jaguar rollator item no. 312410+11+12+13+15

ACCESSORIES LIST

| Item nr. | Accessories item nr. |
|--------------------|----------------------|
| 312410+11+12+13+15 | No accessories |

Harmonized norms used during conformity estimation:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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