Moving Health Forward

**EN** [User Manual](#)

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
# LEJRELET

## Pad Low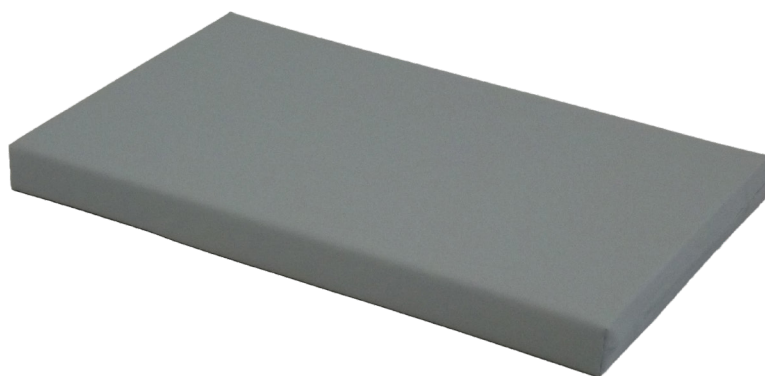

# EN

## Application


LEJRELET Pad Low is made from pressure relieving viscoelastic foam. This makes Pad Low ideal for pressure relief and support in several different positioning situations.

Pad Low can be used flat, bent, folded or coiled.


Pad Low can be used alone or in combination with other LEJRELET products.

Examples for use of LEJRELET Pad Low:


- Support of arm/hand
- Pressure relief of heels
- Pressure relief between knees/ankles
- Coiled, bent or folded i.e. as abduction wedge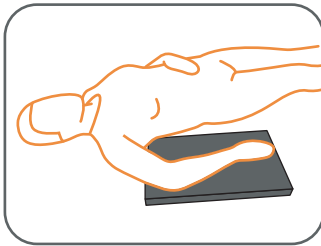


*Support of arm/hand*

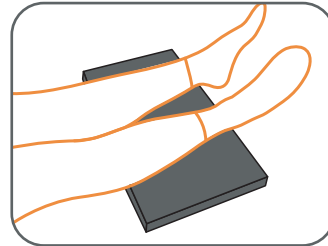*Pressure relief of heel*

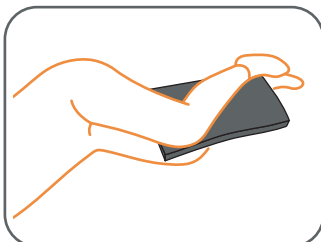*Pressure relief between knees in lateral position*

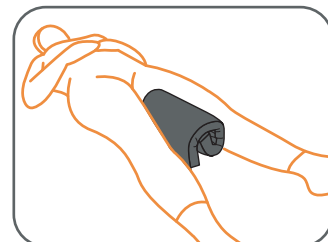*Coiled as abduction wedge*

## Technical Information

### Filling:

LEJRELET Pad Low is made of viscoelastic foam preventing decubitus.

### Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

## Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

**Size:** H 4 x W 30 x L 50

**Item no.:** 5000004

**Weight:** 350 g

Be aware that the patient is positioned such that airways are kept clear.

## Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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**This Page is Left Intentionally Blank**

**This Page is Left Intentionally Blank**

Moving Health Forward

**PRESSURE ULCER PREVENTION**  
**SAFE MOVING & HANDLING**  
**SPECIALIST THERAPIES**  
**BATHROOM SAFETY**  
**RENTAL & SERVICE SOLUTIONS**

Direct Healthcare Group Ltd  
Withey Court, Western Industrial Estate  
Caerphilly, United Kingdom  
CF83 1BF

T: +44 (0) 800 043 0881  
F: +44 (0) 845 459 9832  
E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)

**EC** **REP**

**EU Authorised Representative, Importer, and Distributor:**

Direct Healthcare Group Sverige AB,  
Torshamnsgatan 35, SE-164 40, Kista  
Sweden  
T: +46 (0) 8 557 200

**Australian Sponsor:**

Direct Healthcare Group Pty Ltd.,  
68 Howe Street, Osborne Park,  
Western Australia 6017  
T: +61 (0) 423 852 810

Issue 3  
Date: June 2022

[DHG-HEALTHCARE.COM](http://DHG-HEALTHCARE.COM)