## **WALL MOUNTED FOLDING SUPPORT RAIL**

**Art. 311655, 311676**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions for use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

The folding support rail is designed for use by disabled persons with limited motor skills. It is used as a mobility aid, making it easier to get up and sit down on the toilet, as well as to stabilize the body and safely move around the bathroom for people who require such assistance due to their condition.

### Indications

The product is intended for the elderly people with mobility limitations, wheelchair users, people with limited lumbar, knee or hip mobility, and anyone who has difficulty taking a sitting position and standing up. The user is required to be able to balance independently while sitting and standing, as well as to have sufficient ability to maintain a stable sitting and standing position.

### Contraindications

The product must not be used if the user has balance problems or perceptual disturbances.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

|                                                                 |                                                                              |
|-----------------------------------------------------------------|------------------------------------------------------------------------------|
| Mobilex A/S<br>Grønlandsvej 5<br>DK-8660 Skanderborg<br>Denmark | Phone: +45 87 93 22 20<br>Telefax: +45 87 93 17 77<br>Email: info@mobilex.dk |
|-----------------------------------------------------------------|------------------------------------------------------------------------------|

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the device
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- Do not exceed the maximum user weight capacity and maximum loading force described in Technical Data.
- Before use, the product should be completely assembled and well attached to the wall. Please contact us if there are any problems in this regard
- Folding and adjustable mechanisms can become a cause of danger if body parts get into the gap between components and could become trapped when the gap narrows.
- Always check that the product is securely fastened before each use.
- Use caution on wet or slippery surfaces.
- Use the product only for the purpose intended by the manufacturer.
- Clean the product regularly.
- Always check the stability of the handle before use.

### Parts supplied

The contents of a package are: 1 foldable handle, 1 user manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

The folding support handrail is constructed of durable steel, powder coated with a special white enamel. It consists of a flat bracket, used for mounting, and a bent tube, which is the handrail. The diameter of the tube and its shape makes it easy and safe to use. It has a hinged mechanism that allows it to be raised to an upright position, so it doesn't take up much space when not in use.

## Assembly description

Tools needed for installation: electric drill, level, wrench or screwdriver. Additional necessary materials: 6 screws along with wall plugs, of the type and size suitable for the type of wall.

1. Using a level, determine the position of the handrail on the wall, at a height selected for the height of the person using this aid and in a position suitable for the functionality in the surrounding area. The handrails should be fixed so that the movement of tilting the handrail is vertical and upward.
2. Mark the attachment points and drill 6 holes with a diameter suitable for the expansion bolts suitable for proper installation of the handrail. The expansion bolts should be selected according to the type of substrate (e.g. brick, hollow brick, hollow block, gypsum board, cement bonded tiles).
3. Insert the expansion bolts into the wall. Place the handrail bracket on the holes in the wall, making sure that they are exactly aligned with the handrail bracket. Put the screws through the holes of the handrail bracket and screw them into the expansion bolts.
4. Tighten the screws with a suitable tool and check that the handrail is securely fastened to the wall.

**NOTE:** The selection of materials, method and quality of installation is the responsibility of the person carrying out this installation!

**NOTE:** Make sure that the chosen wall is suitable to support the handrail and the weight of the user.

**NOTE:** Test load the handrail to make sure it is securely attached to the wall before you start using it.

**NOTE:** The installation of the handrail should be carried out by qualified personnel.

## Function description

### Folding the handrail

To use the handrail, lower it down to a horizontal position, and when out of use, lift it back up to the vertical position.

## Accessories

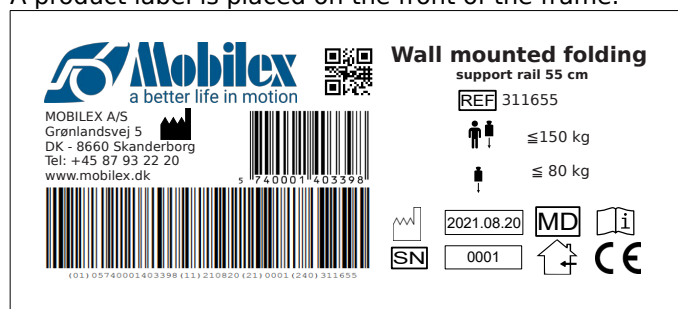
There are no accessories provided for this product.

## 4. Technical data

| Foldable Handrail | Art. 311655         | Art. 311676 |
|-------------------|---------------------|-------------|
| Length            | 55 cm               | 76 cm       |
| Weight            | 2,1 kg              | 2,5 kg      |
| Diameter          | 3,2 cm              |             |
| Max. user weight  | 150 kg              |             |
| Max. load         | 80 kg               |             |
| Material          | Powder coated steel |             |

## Product label

A product label is placed on the front of the frame.

Example of product label

|  |                          |  |                    |
|--|--------------------------|--|--------------------|
|  | QR-code link to homepage |  | Manufacturing date |
|  | Manufacturer             |  | Loading force      |
|  | Observe the user manual  |  | serial no          |
|  | CE marked product        |  | Item Number        |
|  | User weight              |  | Medical Device     |
|  | Only to be used indoor   |  |                    |

Meaning of the icons

## 5. Care and maintenance

Normal dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The product basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General re-tightening of the product

**Recommended service life:** Normally 5 years depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

### **Transportation**

The device is not designed for transportation as it is permanently fixed to the wall.

### **Reuse**

The product is suitable for reuse. Before a new user takes over the product, it must be checked according to point 5.

### **6. Most common problems and solutions**

| <i>Problems</i>                    | <i>Possible causes</i> | <i>Solutions</i> |
|------------------------------------|------------------------|------------------|
| The handrail doesn't move smoothly | The hinge is dirty     | Clean the hinge  |

### **7. Disposal of the product**

The product can not be disposed of with household waste but must be brought to the local recycling center.

### **8. Guarantee terms**

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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**Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, SRN: DK-MF-000021885**
Moblex A/S